

---

## Optimetrics TC-70 Table Clamp Mount, W/ Counterbalanced Phoropter Arm




# “ D A N G E R ”

**Do Not Move ADJUSTABLE LEVER #2c, To RELEASE TENSION, UNTIL ARM IS BEING HELD SECURELY ON TC-70 TABLE CLAMP Mount**



**Do Not Mount PHOROPTER UNTIL TC-70 TABLE CLAMP Mount IS SECURELY CLAMPED IN PLACE**

## **MOUNTING THE SPRING COUNTER BALANCED PHOROPTOR ARM:**

- 1- After attaching TC-70 table clamp stand to desk or table, place the C/B Arm's Pin #2b into the top of the center column tube.
- 2- Make sure spring arm is sitting fully flush on the center column tube
- 3- Loosen knob #3c and swing small arm out from under spring arm
- 4- With one hand pushing/holding the C/B Arm down, turn adjustable lever #2c to release tension.
- 5- Slowly, let complete spring move upwards.
- 6- Counter weight water bag can now be attached
- 7- The Phoropter can now be placed on the Phoropter Pin #3a
- 8- Attach safety screw to your phoropter through safety hole #3b

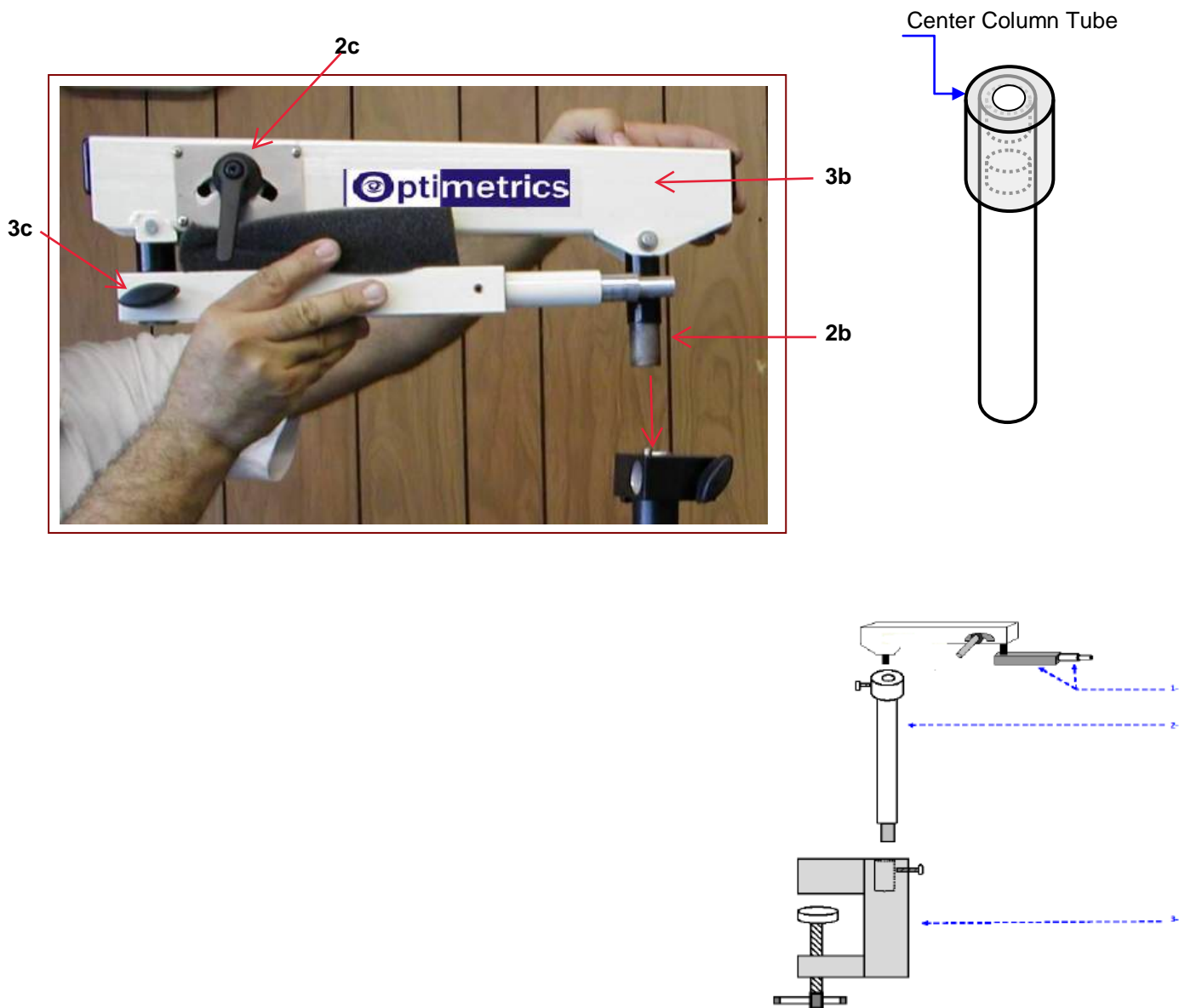

# Optometrics TC-70 Table Clamp Mount, W/Counterbalanced Phoropter Arm

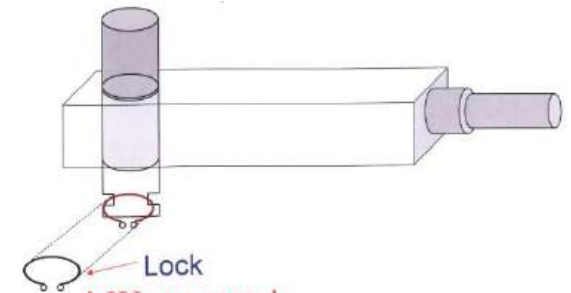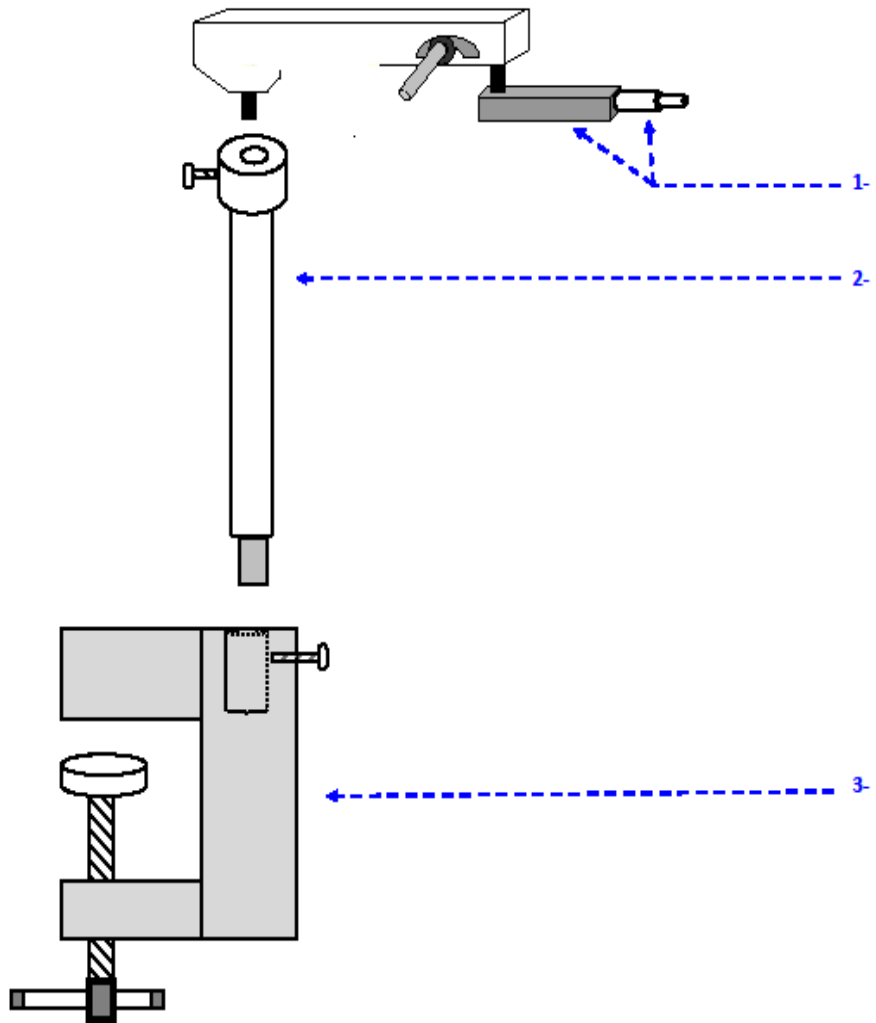

**! WARNING !**

This lock must sit in the groove properly to prevent the phoropter arm from falling off.

**Optometrics**

305-863-4144 Toll Free

1-866-348-3009

[www.optometrics.com](http://www.optometrics.com)